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Glassarm
Fiberglass

Application Information

Glassarm – fiberglass in diameter of 1,1 mm. Suitable for teeth retention and manufacturing adhesive bridge dentures in the anterior and posterior teeth.

Glassarm – very strengthen to explode. The adhesive bridge dentures hold the load up to 250 kg.

Glassarm – applied with silans, adhesive bonds, and flowable composite resin and ormocers.

Field of application:

- retention of pathologically injured teeth;
- strengthening of adhesive bridge dentures.

Instructions for use:

The application of Glassarm is relieved because of good adaptation to the teeth surface and lack of necessity in special implements.

For prevention of blooming of fiber glass Glassarm it is necessary to spend cutting by a method flash-off. Also it is possible to spend cutting by scissors, thus it is necessary to put on a site of the future cut bond, with the subsequent polymerization, with the purpose of prevention of blooming of a fiber.

Teeth-splinting:

Brush the tooth surface where retainer is supposed to be by toothpaste without fluorine. If it is necessary carry out teeth preparation in the retainer location. Preparation is carried out to enforce teeth retention and to supply retainer with area.

Use the primer and bond in combination with total etching technique to achieve the normal adhesion. After teeth preparation etch the whole cavity beginning with enamel edge with etching gel during 40 seconds. Remove etching gel by suction, rinse out the cavity by pure water during 40 seconds, and remove excess of humidity (air jet). The dentin surface must be slightly moist, and enamel surface – chalk-white.

Apply by microbrush thin layer of adhesive bond onto the prepared teeth surfaces. After 40 seconds, thoroughly swell up the bond by thin airjet. Polymerize halogen light for 20 seconds. Do not touch the layer inhibited by oxygen, because it is very important for composite ligament.

Cure Glassarm by silan, after 20 seconds thoroughly remove excess of humidity (air jet). Apply the bond onto the Glassarm and after 20 seconds thoroughly remove excess of humidity (air jet) in the same way. Evenly impregnate Glassarm with flowable composite resin and ormocer. Apply thin layer of flowable composite resin and ormocer to preparation area. Dispose in prepared cavities Glassarm, press it to the bottom of the cavity, and polymerize halogen light.

Restore the cavity by composite resin or ormocer. Carry out final polishing of composite excesses, with the following polishing. The normal polishing is achieved by use of polishing silicons. For contact areas polishing it is better to use special disks.

Manufacturing of adhesive bridge dentures:

Carry out preparation of abutment teeth according to the method you have chosen. During the process of manufacturing adhesive bridge dentures by clinical method carry out the total etching of the preparation surface during 40 seconds. Remove etching gel by suction; rinse out the cavity by pure water during 20 seconds, remove excess of humidity (air jet). The dentin surface must be slightly moist, and enamel surface – chalk-white.

Apply by microbrush thin layer of adhesive bond onto the prepared teeth surfaces. After 40 seconds, thoroughly swell up the bond by thin airjet. Polymerize halogen light for 20 seconds. Do not touch the layer inhibited by oxygen, because it is very important for composite ligament.

Cure Glassarm by silan, remove excess of humidity (air jet). After curing by silan it would be better to put the Glassarm into vacuum for 5 minutes. After this process cure Glassarm by bond, remove excess of humidity (air jet). Impregnate Glassarm with flowable composite (ormocer).

Apply thin layer of flowable composite resin and ormocer onto preparation surface. Put Glassarm into the preparation surface according to the method of manufacturing adhesive bridge dentures you have chosen. Polymerize halogen light. Design adhesive bridge dentures from composite resin (ormocer), polymerize halogen light.

Carry out polishing of composite excesses, with the following polishing. The normal polishing is achieved by use of polishing silicons. For contact areas polishing it is better to use special disks.

Instructions/ precautions:

- side effects are unknown, the material is biologically inert.
- during application avoid the direct contact with fingers and the cavity liquid. During washing by pure water the original characteristics are restored.

Keeping:

Under the temperature of 0 – 40 °C. Sell-by date 5 years. Before application the product must be heat up to the room temperature. Do not use after expiring sell-by date.

Glassarm

Packaging 7 × 140 mm, Ø 1,1 mm.

Glassarm is produced for application in dental care. Verbal or written instructions, and our advice based on the detailed study of the product do not commit you to refuse using of our output products in accordance with your specialization and your discretion. Nevertheless, application of our products without our control is exclusively under your responsibility. Naturally, we guarantee conformity of the products to the active standards, delivery and sale conditions – to the accepted standards.

Phone number: +7 903 5219012

e-mail: mail@glassarm.com

<http://glassarm.com>